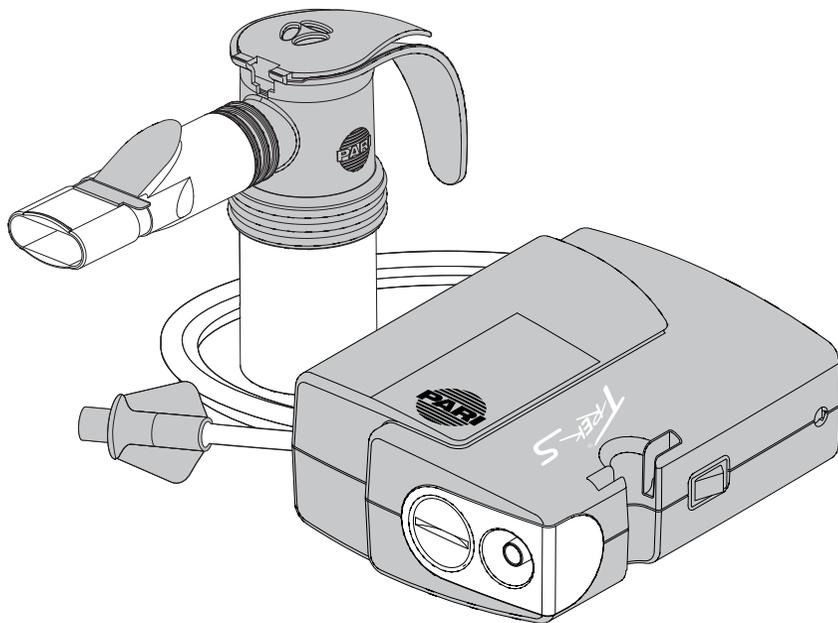
PARI TREK[®] S

COMPACT NEBULIZER SYSTEM

INSTRUCTIONS FOR USE MODE D'EMPLOI

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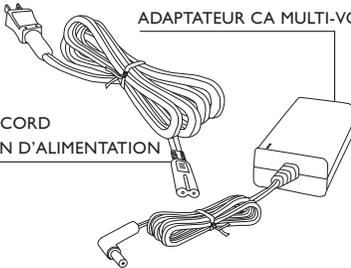
MULTI-VOLTAGE AC ADAPTER
ADAPTATEUR CA MULTI-VOLTS

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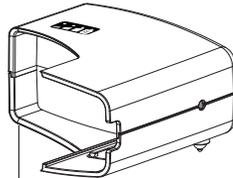
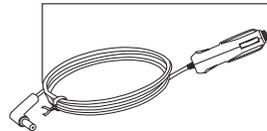
III. PARI TREK® S RECHARGEABLE BATTERY AND 12V DC ADAPTER ILLUSTRATION (OPTIONAL PORTABLE POWER KIT)

1. Battery Level Indicator
2. Charge Indicator
3. Charge Receptacle
4. Release Button
5. 12V DC Adapter
6. Battery Contacts (*not visible in illustration*)

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
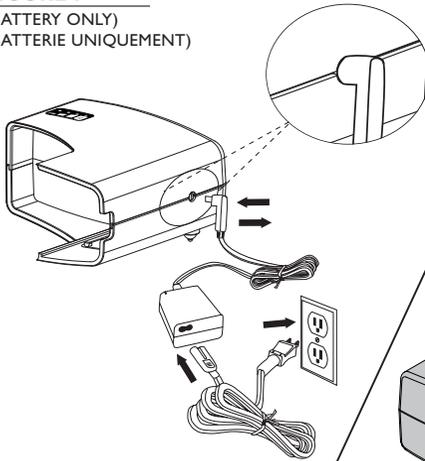
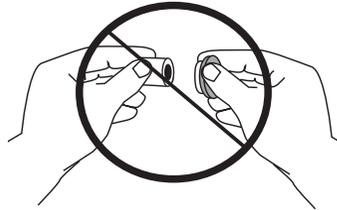
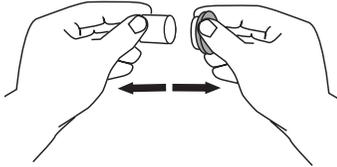
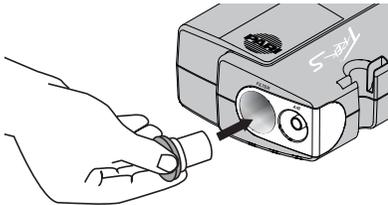
5

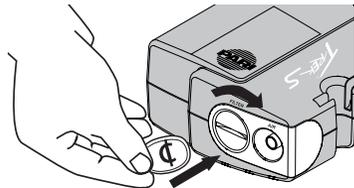INSTRUCTIONS FOR USE

I.	PARI TREK® S COMPRESSOR SYSTEM COMPONENTS.....	1
II.	PARI TREK® S COMPRESSOR ILLUSTRATION.....	2
III.	PARI TREK® S BATTERY & 12 V DC ADAPTER ILLUSTRATION.....	3
	Other Illustrations.....	4-6
IV.	INTRODUCTION.....	8
V.	SAFETY PRECAUTIONS	
	Compressor Safety Precautions.....	9
	Nebulizer Safety Precautions.....	11
	Battery Safety Precautions.....	12
VI.	GETTING STARTED.....	13
VII.	COMPRESSOR	
	Set Up.....	13
	Operation From a Wall Power Source.....	14
	Operation Using 12 VDC Adapter.....	14
	Operation Using the Rechargeable Battery.....	14
VIII.	NEBULIZER	
	Set Up.....	16
	Treatment.....	16
	Cleaning.....	17
	Disinfecting and Sterilizing.....	17
IX.	RECHARGEABLE BATTERY	
	Charging.....	18
	Storing.....	18
X.	PARI TREK® S COMPRESSOR	
	Maintenance.....	19
	Specifications.....	20
	Limited Warranty.....	22
	Return Procedure.....	23

PARI TREK® S COMPRESSOR INSTRUCTIONS FOR USE

IV. INTRODUCTION

These instructions contain important information and safety precautions on the PARI TREK® S Compressor Nebulizer System.

- Illustrations of the PARI TREK® S Compressor System Components are on **Page 1**.
- Illustrations of the PARI TREK® S Compressor are on **Page 2**.
- Illustrations of the Rechargeable Battery and 12V DC Adapter for the PARI TREK® S Compressor are on **Page 3**.
- The illustrations to view while reading the instructions are on **Pages 4-6**.

Before using your PARI TREK® S Compressor Nebulizer System, you should read and understand the entire Instructions For Use for the PARI TREK® S Compressor, PARI Reusable Nebulizer and optional rechargeable battery. Take special note of all safety precautions marked **DANGER** and **WARNING**. Save these instructions for future reference.

V. SAFETY PRECAUTIONS

IMPORTANT SAFEGUARDS

Read all warnings before using your PARI TREK® S compact nebulizer system, rechargeable battery and 12V DC adapter.

WARNING

Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication. Read all warnings and instructions before use. Use the product only for its intended use as described in these Instructions for Use. Improper use can result in serious or fatal illness/injury, improper treatment or property damage.

The Trek® S Compact Nebulizer System is a portable, DC-powered compressor intended to provide a source of compressed air for use with jet nebulizers. The Trek® S is intended for adult and pediatric patients. Use only with nebulizers, tubing and accessories authorized by PARI for this device, for example, the PARI LC® Plus or LC® Sprint Reusable Nebulizer. Please refer to the compressor specifications on page 18 to ensure that any other components are compatible with this system. Other nebulizers, tubing and accessories may not be compatible and/or result in improper treatment. Use with other tubing, nebulizers and accessories should be reviewed with your healthcare professional.

COMPRESSOR SAFETY PRECAUTIONS

DANGER

To reduce the risk of serious or fatal injury from electrocution:

1. Do not place or store the product where it can fall or be knocked into bathtub, sink, water, or other liquid. Do not place or drop into water or other liquid. Do not use while bathing.
2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.
3. Always unplug immediately after using.

WARNING

To reduce the risk of serious or fatal injury from electrocution, fire, burns and to reduce the risk of damage and malfunction of the unit:

1. Before use, check the serial number label on the bottom of the compressor, battery and on the Multi-Voltage AC Adapter to ensure that the voltage and current indicated on the unit correspond with the voltage and current available.

2. Never operate this product if it:

- Has a damaged cord or plug
- Is not working properly
- Has been dropped or damaged
- Has been exposed to any liquids inside the case

Return non-operating product to an authorized PARI Service Center for examination, repair or replacement.

3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or use extension cords.

4. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.

5. Never drop or insert any object into any openings.

6. Be sure the PARI TREK® S Compressor and the Multi-Voltage AC Adapter are unplugged prior to cleaning.

7. Do not open or disassemble the PARI TREK® S Compressor. Refer all servicing to an authorized PARI Service Center.

8. Use only adapters that are authorized for the PARI TREK® S Compressor. Use of unapproved accessories can lead to improper treatment or damage to the compressor, adapters, or the battery.

9. Never spray liquids onto the compressor housing or battery. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluids enter the unit, return the product to an authorized PARI Service Center for examination and repair.

10. Do not use the PARI TREK® S Compressor or the PARI Reusable Nebulizer while operating a vehicle.

11. Portable and mobile RF communication devices can disrupt electrical medical equipment. The electrical field generated when the compressor is running can interfere with screens and monitors (flickering). For this reason, the compressor must be placed far enough away from screens/monitors to prevent such interference. Technical electromagnetic compatibility data is available in table form upon request from PARI Respiratory Equipment Inc. / Patient Connection **1.800.FAST.NEB (327.8632), press 2.**

NEBULIZER SAFETY PRECAUTIONS

WARNING

1. Nebulizer is for single patient use (one person) only. Do not share your nebulizer with other patients.
2. To reduce the risk of increased bacterial growth, infection, illness, or injury from contamination, thoroughly clean and dry all parts of the nebulizer and thoroughly dry any moisture or condensation in the tubing at the end of every treatment, following the Instructions for Use that are enclosed with the PARI Reusable Nebulizer.
3. Cleaning the nebulizer after each treatment is not enough, you must also regularly disinfect or sterilize the nebulizer between treatments to prevent serious or fatal illness caused by contamination of the nebulizer.
4. Never use a nebulizer with a clogged nozzle. If the nozzle is clogged, the aerosol mist will be reduced, altered or stopped, which will diminish or eliminate the effectiveness of the treatment. If clogging occurs, replace with the same model PARI Reusable Nebulizer.
5. Close supervision is necessary when this product is used by or near children and physically or mentally impaired persons. Never use when sleeping or drowsy.
6. This product contains small parts that may present a choking hazard to small children. The tubing also presents a strangulation hazard. **ALWAYS USE CLOSE ADULT SUPERVISION WHEN ADMINISTERING A TREATMENT TO A CHILD. DO NOT LEAVE CHILD UNATTENDED DURING TREATMENT.**
7. Children under the age of 3 or any patient who is unable to use a mouthpiece properly under supervision should use a face mask.
8. Before use, check PARI Reusable Nebulizer for proper assembly. All parts should be seated firmly in place. Use of an improperly assembled nebulizer could diminish or prevent adequate delivery of medication and the effectiveness of the treatment.

BATTERY SAFETY PRECAUTIONS

WARNING

1. Use only the PARI TREK® S Rechargeable Battery described in these **Instructions For Use** for operating your PARI TREK® S Compressor.
2. Do not immerse the PARI TREK® S Rechargeable Battery in water.
3. Do not touch the battery with wet or moist hands.
4. Do not expose battery to direct sunlight, humidity, extreme temperatures or chemicals.
5. Make sure battery poles do not come into contact with keys or other metal objects, causing a “short circuit”.
6. To charge the PARI TREK® S Rechargeable Battery use only the PARI TREK® S Multi-Voltage AC Adapter or optional 12V DC Adapter. Using other adapters may damage the battery.
7. In case of failure in charging or operating, unplug the Rechargeable Battery from the Multi-Voltage AC Adapter and turn the unit off. Should the Rechargeable Battery fail to charge or operate properly, even if the Instructions For Use are followed, notify the Authorized Service Center by calling PARI Patient Connection toll free (USA) at **1.800.FAST.NEB (327.8632)**, then **press 2**.
8. Do not attempt to open the plastic case or to replace any components. Do not disassemble Rechargeable Battery. The Rechargeable Battery is a sealed unit and can not be repaired.
9. Protect the environment: when replacing the Rechargeable Battery, recycle old battery or dispose of properly in accordance with local laws.
10. Do not incinerate (burn).

SAVE THESE INSTRUCTIONS

VI. GETTING STARTED

1. Check the box to make sure the following basic components of the PARI TREK® S Compressor Nebulizer System are enclosed:

- PARI TREK® S Compressor
- Multi-Voltage AC Adapter
- Power Cord
- PARI Reusable Nebulizer with Wing Tip™ Tubing
- Carrying Case
- PARI TREK® S Compact Nebulizer System Instructions For Use
- Patient Instructional DVD
- 12V DC Adapter

Refer to **Page 1** to view component illustrations.

If you did not receive the components listed, please contact the dealer where you purchased your PARI TREK® S Compact Nebulizer System.

2. It is a good idea to keep an extra PARI Reusable Nebulizer (part no. **023F35** for PARI LC® SPRINT) and an extra PARI TREK® S Compressor Air Filter (part no. **041E4851P12**) on hand at all times. You may need them if parts become lost, damaged, or soiled.

VII. COMPRESSOR

COMPRESSOR SET UP

1. Place the PARI TREK® S Compressor on a stable flat surface. Place the unit in an area that is clean and free of dust.

NOTE: Check the filter monthly or every 10-12 treatments and replace immediately if dirty, clogged, or wet to ensure proper operation of the compressor.

OPERATION FROM A WALL POWER SOURCE

1. Attach the Power Cord to the Multi-Voltage AC Adapter. Plug the other end of the Power Cord into the wall outlet. The Power Indicator on Multi-Voltage AC Adapter will illuminate green. Next, plug the Multi-Voltage AC Adapter into the Power Receptacle on the side of the PARI TREK® S Compressor. **Refer to Page 4 Figure A.**
2. The PARI TREK® S Multi-Voltage AC Adapter automatically adjusts to the incoming voltage (100-240 V) so that it can be used in most countries. For use internationally, plug pattern adapters may be required to plug the Power Cord into a wall outlet. See voltages listed on the Multi-Voltage AC Adapter label for more specific information.

OPERATION USING 12V DC CAR ADAPTER

NOTE: Do not use the PARI TREK® S Compressor or the PARI Reusable Nebulizer while operating a vehicle.

1. Plug the 12V DC Adapter into the vehicle outlet and plug the other end of the adapter into the Power Receptacle on the side of the PARI TREK® S Compressor or optional Rechargeable Battery Pack. **Refer to Page 4 Figure B.**

OPERATION USING THE RECHARGEABLE BATTERY (optional accessory)

1. Completely charge the battery before using for the first time for more than two hours or until all three battery level indicators are lit, and the charge indicator turns off. **Refer to Charging the Battery on Page 18.**
2. The Rechargeable Battery can not be connected directly to the PARI TREK® S Compressor if either the Multi-Voltage AC Adapter or 12V DC Adapter are plugged into Power Receptacle of the PARI TREK® S Compressor. **Refer to Page 4 Figure C Number 1.**
3. Attach the Rechargeable Battery to the PARI TREK® S Compressor by placing the Battery onto the rear section of the PARI TREK® S Compressor and sliding it into place. **Refer to Page 4 Figure C Number 1.** Ensure battery is securely locked onto the compressor before use.

4. Use the Battery Indicator lights to determine the status of the Battery: **(Refer to Battery Illustration on Page 3.)**

Charging:

Charge LED "ON" - Battery is being charged.

Charge LED "OFF" - Battery is fully charged.

Operation With Battery:

Three Green Lights - Battery has more than 75% remaining battery life.

Two Green Lights - Battery has more than 50% remaining battery life.

One Green Light - Battery has more than 25% remaining battery life.

One Green Light flashing - Less than 25% of the battery life is remaining.

When the battery is completely discharged all indicators on the battery will extinguish and the compressor will shut off. **Refer to Charging the Battery on Page 18.**

Note: *If battery fails to operate, connect the Power Cord into the Multi-Voltage AC Adapter and plug the other end of the Power Cord into the wall outlet. Plug the other end of the Multi-Voltage AC Adapter into the Charge Receptacle on the side of the Rechargeable Battery. The Charge Indicator should turn "ON" indicating that the battery has re-set and is now charging. If the Charge Indicator does not appear, or if all four indicators are blinking, the battery has failed and should not be used. Please contact PARI Patient Connection at **1.800.FAST.NEB (327.8632) and press 2.***

5. To remove the Battery from the PARI TREK® S Compressor, push and hold the Release Button down **(2-A)** on the underside of the Rechargeable Battery then slide the Battery from the compressor **(2-B)** and remove from the compressor. **Refer to Page 4 Figure C, Numbers 2-A and 2-B.**

6. The Rechargeable Battery can be recharged while it is attached to the PARI TREK® S Compressor. **Refer to Page 5 Figure F.**

VIII. NEBULIZER

NEBULIZER SET UP *(Note: Nebulizer is not sterile)*

1. Thoroughly read and follow the Nebulizer Set Up in the Instructions for Use that are enclosed with the PARI Reusable Nebulizer.

2. WING TIP™ TUBING MUST ALWAYS BE USED TO ATTACH NEBULIZER TO PARI TREK® S COMPRESSOR. Refer to Page 5, Figure D.

NEBULIZER TREATMENT

1. Take your treatment as directed by your physician.

2. The PARI Reusable Nebulizer has two valves that open and close during the nebulizer treatment when the Mouthpiece is used. These valves help to maximize aerosol delivery and minimize medication waste.

3. Sit in a relaxed, upright position and press the black On/Off Switch (**Refer to Page 2**) to turn on the PARI TREK® S compressor.

4. Place the Mouthpiece on top of your tongue and close your lips around the Mouthpiece. Take slow deep breaths to inhale the aerosol medication slowly. **Refer to Page 5 Figure E.** As you inhale, the valve on the Inspiratory Valve Cap opens.

5. Exhale slowly through your mouth. As you exhale, the Expiratory Valve on the Mouthpiece opens, releasing your exhaled aerosol. At the same time, the Inspiratory Valve closes on the top of the nebulizer which reduces waste.

6. Continue steps 4 and 5 until the medication is gone or you hear a “sputtering” noise. Press the black On/Off Switch to turn off the PARI TREK® S Compressor.

The PARI TREK® S Compressor is protected by a thermal switch. If the PARI TREK® S Compressor has reached its temperature limit it will automatically turn off to prevent damage to the PARI TREK® S Compressor. In the unlikely event this occurs, the PARI TREK® S Compressor may need approximately 30 minutes to cool down prior to resuming the treatment.

Switch off the the PARI TREK® S Compressor and let it cool down before restarting. If the PARI TREK® S Compressor does not start after it is switched on after cooling for 30 minutes, please allow another few minutes for the compressor to cool and try again.

TREATMENT TIMES

- Patients who use a 2.5ml volume fill will typically find that the PARI TREK® S Compressor will deliver the treatment in less than 10 minutes. Treatment time depends upon breathing volume, patient's age and disease state.
- The PARI TREK® S Compressor was designed for intermittent use to deliver aerosol medications, and not for continuous use humidification or prolonged treatments (not over 30 minutes at one time). The PARI TREK® S Compressor is equipped with a resettable thermal protector to prevent damage to the PARI TREK® S compressor unit. If the thermal protector cuts off, the PARI TREK® S Compressor will shut down until it is cool (approximately 30 minutes), then treatments can continue.
- If you will be using the PARI TREK® S Compressor for treatments longer than 30 continuous minutes at one time, please contact PARI Patient Connection at 1.800.FAST.NEB (327.8632) (USA), then press 2 for tips on prolonged use.

CLEANING THE PARI REUSABLE NEBULIZER

I. Thoroughly read and follow the Cleaning section in the Instructions for Use that are enclosed with the PARI Reusable Nebulizer.

DISINFECTING AND STERILIZING THE PARI REUSABLE NEBULIZER

I. Thoroughly read and follow the Disinfecting and Sterilizing sections in the Instructions for Use that are enclosed with the PARI Reusable Nebulizer.

IX. RECHARGEABLE BATTERY

CHARGING THE BATTERY *(optional accessory)*

1. The PARI TREK® S Battery must be charged for more than two hours before using it the first time. Refer to PARI TREK® S Battery Illustration on Page 3.
2. To charge the Battery, connect the Power Cord into the Multi-Voltage AC Adapter and plug the other end of the Power Cord into the wall outlet. Plug the other end of the Multi-Voltage AC Adapter into the Charge Receptacle on the side of the Rechargeable Battery. **Refer to Page 5 Figure F.** The Charge Indicator will turn “ON” indicating the Battery is charging.
3. When the Rechargeable Battery is fully charged, the charge Indicator will turn “OFF” indicating the Battery is fully charged. Unplug the Multi-Voltage AC Adapter from the Charge Receptacle on the side of the Rechargeable Battery. The amount of time it takes to recharge the Battery will depend on the age and charge level of the Battery.

NOTE: *Do not leave the Multi-Voltage AC Adapter plugged into the Rechargeable Battery for more than 7 days. Leaving the battery connected to the power adapter for long periods of time may decrease the life of the battery.*

4. When not regularly using the Battery, the Battery should be fully charged every six months to optimize and preserve the life of the Rechargeable Battery.

STORING THE RECHARGEABLE BATTERY *(optional accessory)*

1. The Battery should be stored fully charged in a cool dry place.
2. Battery should be fully charged every six months.

NOTE: *If the Battery goes for a long period of time without charging, Battery life will be decreased.*

X. PARI TREK® S COMPRESSOR

MAINTENANCE

AIR FILTER: The Air Filter on the front panel of the PARI TREK® S Compressor is bright white when new. In time, the Air Filter will turn gray, which means it is time to replace it. You must replace the filter immediately if it gets wet or becomes clogged. The Air Filter lasts up to 500 hours of use before replacement. A good general rule is to replace the Air Filter whenever you replace your PARI Reusable Nebulizer (every 6 months). To remove the Air Filter, make sure the PARI TREK® S Compressor is turned off. Use a coin to turn the Air Filter cover counter clockwise and pull the cap and filter out and remove the filter from the cap. **Refer to Page 6 Figure G.** To replace filter, press the new filter into the cap with the open end visible. Place the cap and filter back into the front of the compressor and using a coin turn the filter cover clockwise to tighten as illustrated on **Page 6 Figure G.**

NOTE: Do not attempt to wash and reuse Air Filters. Do not use substitute filter material. Compressor damage may result.

CLEANING THE PARI TREK® S COMPRESSOR HOUSING: Use a clean, damp cloth to wipe the exterior of the PARI TREK® S Compressor housing. Be careful not to get liquids onto the Power Receptacle or the Battery Contacts since damage could occur.

MAINTENANCE AND REPAIR: Never operate the PARI TREK® S Compressor if it is improperly or incompletely assembled, or damaged. Use only original PARI parts and attachments authorized by PARI for the PARI TREK® S Compressor. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time.

Note: There are no serviceable parts inside the PARI TREK® S Compressor case. Service by anyone voids warranty.

CLEANING THE RECHARGEABLE BATTERY: Use a clean, dry cloth to wipe the exterior of battery. Be careful not to get liquids onto the Charge Receptacle or the Battery Contacts since damage could occur.

PARI TREK® S COMPRESSOR NEBULIZER SYSTEM SPECIFICATIONS

Weight: (Compressor only).....	0.8 lb (360 grams)
With Rechargeable Battery:.....	1.2 lbs (520 grams)
Dimensions: (Compressor only).....	4.5" x 3.5" x 1.75"
Certification:.....	IEC/EN 60601-1 and IEC/EN 60601-1-2

Compressor Performance:

Maximum Pressure of Compressor:.....	40 PSI (2.75 Bar)
Maximum Liter Flow of Compressor:.....	8 LPM
Operating Pressure of Compressor ¹ :.....	14.5 PSI (1.0 Bar)
Operating Liter Flow of Compressor ¹ :.....	4.2 LPM

¹Measured with PARI LC® Sprint or LC® Plus Reusable Nebulizer attached.

Ambient Operation:

Temperature Range:.....	50°F to 104°F (10°C to 40°C)
-------------------------	------------------------------

Transport/Storage:

Ambient Temperature:.....	-13°F to 158°F (-25°C to 70°C)
Humidity:.....	10% to 95%
Atmospheric Pressure:.....	500 hPa - 1060 hPa
Compressor Electrical Requirements:.....	Input 12VDC 1.67 A 18 watts

Optional Rechargeable Battery:

Battery Type:.....	Lithium Ion (Li-Ion)
Operating time with battery:.....	Approximately 50 minutes
Number of treatments per charge ² :.....	Approximately 8 treatments
Normal time to charge:.....	Approximately 2 hours

²Using a PARI Reusable Nebulizer to nebulize 2.5 ml of medication.

Electrical Requirements:

Input:.....	12VDC 1.67 A
Output:.....	12VDC 1.67 A

Multi-Voltage AC Adapter:

FUHUA Model #UE20-120167SPA I (or equivalent)

Input Voltage:.....100-240 VAC 50/60Hz 0.5-0.25 A

Output Voltage:.....12 VDC 1.67 A

Explanation of Symbols:

ATTENTION: Read all “Instructions For Use” before operation:..... 

Insulation enclosed Class II equipment:..... 

Type “B” protection from electrical shock:..... 

Do not dispose of in unsorted municipal trash.

Please consult local laws and regulations for proper disposal:..... 

Power Switch:..... 

”On” position (compressor is running):..... —

”Off” position (compressor is not running):..... 

Note: When switch is in the “Off” position and the compressor is not running, there will still be electrical potential inside the compressor housing. To remove power from the compressor, the power plug or battery pack must be removed.

Maintenance and Repair: Never operate the PARI TREK® S Compressor if it is improperly or incompletely assembled, or damaged. Use only original PARI parts and attachments authorized by PARI for the PARI TREK® S Compressor. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time.

Note: There are no serviceable parts inside the PARI TREK® S Compressor case. Service by anyone voids warranty.

PARI TREK® S COMPRESSOR LIMITED WARRANTY

PARI offers a standard three (3) year warranty on the PARI TREK® S Compressor and six (6) months on accessories. The warranty period will be printed on the serial number label located on the bottom of the compressor. PARI warrants that the PARI TREK® S Compressor and its parts will be free from defects in workmanship and materials for a period of thirty-six (36) months from the date of first retail purchase of the compressor and six (6) months on the PARI Reusable Nebulizer and Li-Ion Rechargeable Battery. This warranty does not cover failures of the tubing or accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service; parts or cleaning solutions, or failure to comply with **Instructions For Use**.

If the warranted equipment should fail during the warranty period, PARI will replace the equipment or parts at issue, provided the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of replacement of the warranted items only. To make a claim under this warranty, contact PARI Patient Connection at: **1.800.FAST.NEB (327.8632), Press 2**. THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PARI TREK® S COMPRESSOR AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THIRTY-SIX (36) MONTHS FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply you.

The sole and exclusive remedy under this warranty is to replace the PARI TREK® S Compressor and/or its parts by PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other rights which vary from State to State.

RETURN PROCEDURE

1. Contact PARI Patient Connection at **1.800.FAST.NEB (327.8632)**, **Press 2** for a customer service representative.
2. In writing, describe the malfunction or symptoms of the PARI TREK® S Compressor. Include your full name, address and daytime telephone number.
3. Send written description and PARI TREK® S Compressor in a protective box to the address provided by Patient Connection.

Specialists in effective inhalation

PARI Respiratory Equipment, Inc. | 2943 Oak Lake Blvd., Midlothian, VA 23112-3998 USA
Tel: 1.800.FAST.NEB (327.8632) and 1.804.253.7274 | Fax: 1.800.727.4112 | www.PARI.com

Patents pending

© 2008 PARI Respiratory Equipment, Inc. All rights reserved.

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© 2008 PARI Respiratory Equipment, Inc. Tous droits réservés.